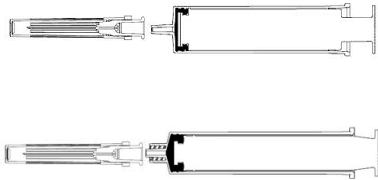
HYPODERMIC AD/RUP SYRINGE (WITH/WITHOUT NEEDLE)

Product Type

AD/RUP syringe with/without needle	Needle Size	Basic UDI-DI: 611800222ADRUPVZ
Luer 3 parts Syringe 1 ml, 3 ml, 5 ml, 10 ml, 20 ml	18G 21G 22G	
	23G 24G 25G	
	28G 29G 30G	

Intended Use

• AD Sterile Hypodermic Syringe for single use intended for the aspiration of fluids or the injection immediately after filling, it is most commonly used for the administration of medications or for blood sampling.

Product Components

Name	Raw material
Barrel	PP - Polypropylene
Plunger	PP - Polypropylene
Gasket	Thermoplastic Elastomer, Latex free
Hypodermic Needle	Stainless Steel, SS304, PP - Polypropylene
Packaging	Medical grade paper and transparent film for EtO (individually blister packed)


HYPODERMIC AD/RUP SYRINGE (WITH/WITHOUT NEEDLE)

Instructions for Use


DON'T PUSH THE PLUNGER FORWARD BEFORE FILLING THE MEDICATION.

Step 1: Open the pack and take out the syringe.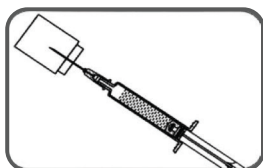


Step 2: Insert the needle into the vial and withdraw the fluid/medication.

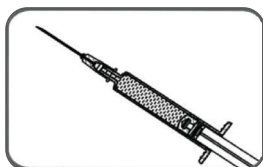Step 3: Flick the barrel to move any air bubbles to the top.
Push the plunger to the desired dosage and expel the air bubbles.


Step 4: Inject the medication. For a 0.5 ml AD syringe, the extended collar of the barrel limits the injection depth to the dermis (the top layer of the skin) and does not go beyond it. After withdrawing the needle, a raised wheal should be present.

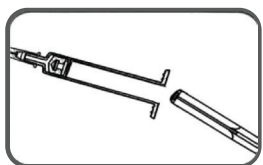Step 5: Continue with the normal injection procedure, including aspiration to ensure the vein is not punctured. If the plunger is pulled back after the injection, it will break.

Step 6: For reconstitution, after Step 3, pull the plunger back fully, keeping the needle pointing downward to draw air into the syringe.

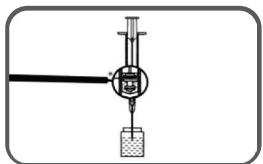Step 7: Release the total fluid into the injection vial, keeping the vial downward. Then, withdraw the medication as in Steps 4, 3, 2, and 5.

Note: The plunger must not cross the ring, or the AD (Auto Disable) mechanism will activate.

Step 8: Dispose of the syringe in a sharps container.

The electronic version of "IFU/ADRUP/001 - Instructions for use" is available at www.immedicare.ma.

HYPODERMIC AD/RUP SYRINGE (WITH/WITHOUT NEEDLE)

Compatibility with other medical devices

Suitable for use with connectors "EN ISO 80369-7 Small-bore connectors for liquids and gases in healthcare applications - Part 7: Connectors for intravenous or hypodermic applications".

Indications

- Hypodermic syringes with needle are devices used to administer medical products and liquids.
- Syringes are designed to be used for intravenous, intramuscular, subcutaneous and taking blood samples.
- The product shelf life is 5 years.

Contraindications

- Hypodermic needles are not used in more than one patient. The product can be used in a one-time sterile condition with a single patient.
- Cleaning or re-sterilization cannot be performed. It is not intended for use outside the specified intended use. After usage, dispose of the product as medical waste.

Cautions

- Do not use the product if the package is opened and damaged.
- Visually inspect that the content of package is complete and the package is undamaged.
- Do not use the product after the expiration date printed on the primary packaging.
- After injection or aspiration, dispose of the product as medical waste.
- The product is for single use only. It cannot be reused or re-sterilized.
- Do not touch the needle tip to prevent contamination/needle sticks. Skin puncture with a contaminated needle may result in serious illness such as hepatitis, AIDS and known/unknown diseases.
- If the sterile packaging is not used immediately after opening, there is a risk of contamination. Product sterility cannot be maintained, do not use the product.
- Reuse of the product may cause infections, cross-contamination and sepsis.
- Sterile syringes are designed to be used immediately after filling, so do not store the medicine for a long time.
- If the wrapping or cap on the end of the syringe is broken or missing, discard it in your sharps container and use a new needle and syringe.

Storage Conditions

- Store in a cool and dry place.
- Keep it away from direct sunlight and moist environment.
- Do not place more than 5 successive boxes on the top of each other.
- The shelf life of product is 5 years.

HYPODERMIC AD/RUP SYRINGE (WITH/WITHOUT NEEDLE)

Description of Symbols Used

	Manufacturer		Fragile. Handle with care
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
LOT	Batch code		Upper limit of temperature (30°C)
REF	Catalogue number		Humidity limitation (35% – 65%)
MD	Medical Device		Do not use if package is damaged and consult instruction for use
UDI	Unique device identifier		Single sterile barrier system
EC REP	European Authorized Representative		Do not reuse
STERILE EO	Sterilized using ethylene oxide		Latex free
	Do not re-sterilize		Non-pyrogenic
	Caution		Consult instruction for use
	Re-use prevention		

Manufacturing Site

IM MEDICARE

Address: Lot 21, Parc Industriel Sidi Bou Othmane, 43303, Morocco

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