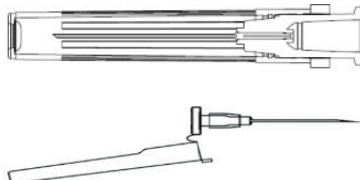
HYPODERMIC NEEDLE

Product Type

Hypodermic Needle								Basic UDI-DI : 611800222HYPONDLEL
16G	17G	18G	20G	21G	22G	23G	24G	
25G	26G	28G	29G	30G	31G	32G		

Intended Use

• Hypodermic needles are used to inject substances into the body or extract fluids from the body while connected to a noninvasive device (e.g., syringe, secondary medication set). A hypodermic needle offers a fast, safe & easy option for rapid delivery of liquids to the patient when the injected substance cannot be ingested orally, as it would not be absorbed or would harm organs.

Product Components

Name	Raw material
Hypodermic Needle	Stainless Steel, SS304, PP - Polypropylene
Protective/Safety Cap	PP - Polypropylene
Packing Material	Medical grade paper and transparent film for EtO (individually blister packed)

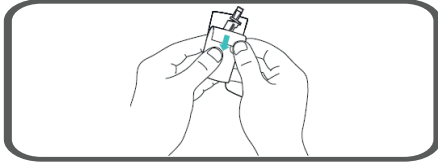
Instructions for Use

- Open the packaging by stripping it from the opening place indicated by the arrow sign.
- Visually inspect that the content of package is complete and the package is undamaged.
- The product packaging consists of a sterile needle.
- Without removing the protective cap of the needle, firmly secure the syringe and needle together, pushing the needle toward the syringe tip and turning it clockwise.
- Carefully remove the protective cap of the needle without damaging the needle. For safety hypodermic needles, carefully remove the safety cap of the needle without damaging the needle.
- Inject the liquid to be injected into the intravenously, intramuscularly, subcutaneously or intradermally, according to the application procedure.


The electronic version of "IFU/HYPOND/001 - Instructions for use" is available at www.immedicare.ma.

HYPODERMIC NEEDLE


Hypodermic Safety Needle - Illustrated Instructions

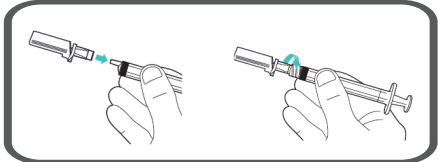
Step 1: Peel open the blister pack and remove the safety device.

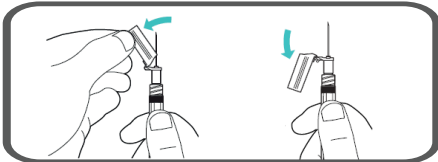
Step 2: Push the safety device onto a standard luer slip connector then screw the safety device onto a standard luer lock connection

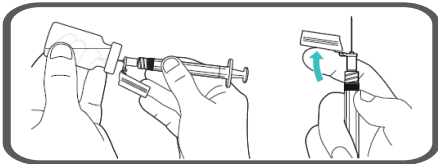
Step 3: Pull back the hinged cover to expose the needle.

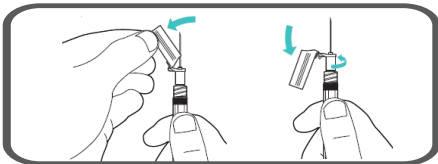
Step 4: Fill the syringe with medication and close the needle cover to transport the needle safely.

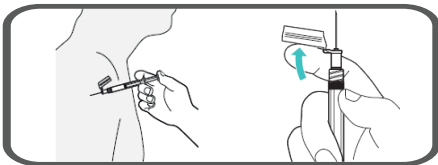
Step 5: Prior to the injection, open the needle cover. The needle cover can be rotated for ease of use.

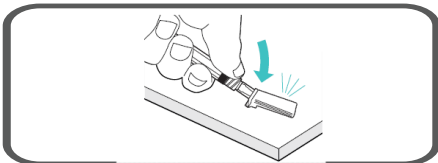Step 6: Administer the injection and close the needle cover immediately after use.

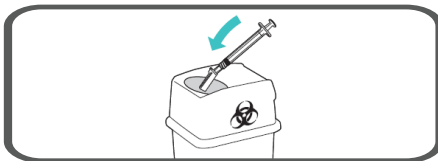Step 7: To activate the locking mechanism press the needle cover against a hard surface.
You will hear a 'click' when the needle cover is locked.

Step 8: Dispose of the used syringe into a sharps container.

HYPODERMIC NEEDLE

Compatibility with other medical devices

Suitable for use with connectors "EN ISO 80369-7 Small-bore connectors for liquids and gases in healthcare applications - Part 7: Connectors for intravenous or hypodermic applications".

Indications

- Hypodermic needles made of empty, sterile and single use stainless steel are used by end users for the injection or withdrawal of both body fluids (blood, etc.) and medical fluids such as medicines.
- It may also be used for liquid/drug preparation.
- The shelf life of product is 5 years.

Contraindications

- Hypodermic needles are not used in more than one patient. The product can be used in a one-time sterile condition with a single patient.
- Cleaning or re-sterilization cannot be performed. It is not intended for use outside the specified intended use. After usage, dispose of the product as medical waste.

Cautions

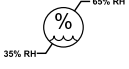
- Do not use the product if the package is opened and damaged.
- Visually inspect that the content of package is complete and the package is undamaged.
- Do not use the product after the expiration date printed on the primary packaging.
- Before use, check that the hypodermic needle is not blocked and the flow is not obstructed.
- For safety hypodermic needles, activate it away from yourself and others with the one-hand technique. For a high level of safety, use only the finger pad area on the large rough tissue when activating the safety cap. If liquid remains on the needle during the activation of the safety cap or after the injection, it may cause a small amount of splash.
- For safety hypodermic needles, to help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use.
- After injection or aspiration, dispose of the product as medical waste.
- The product is for single use only. It cannot be reused, rework or re-sterilized.
- If the sterile packaging is not used immediately after opening, there is a risk of contamination. Product sterility cannot be maintained, do not use the product.
- Reuse of the product may cause infections, cross-contamination and sepsis.
- If any serious incident occurs, it should be reported to the manufacturer and the competent authority

Storage Conditions

- Store in a cool and dry place.
- Keep it away from direct sunlight and moist environment.
- Do not place more than 5 successive boxes on the top of each other.
- The shelf life of product is 5 years.

HYPODERMIC NEEDLE

Description of Symbols Used

	Manufacturer		Fragile. Handle with care
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
LOT	Batch code		Upper limit of temperature (30°C)
REF	Catalogue number		Humidity limitation (35% – 65%)
MD	Medical Device		Do not use if package is damaged and consult instruction for use
UDI	Unique device identifier		Single sterile barrier system
EC REP	European Authorized Representative		Do not reuse
STERILE EO	Sterilized using ethylene oxide		Latex free
	Do not resterilize		Non-pyrogenic
	Caution		Consult instruction for use

Manufacturing Site

IM MEDICARE

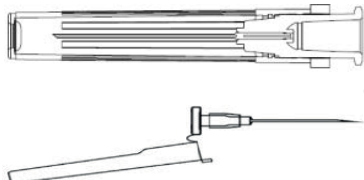
Address: Lot 21, Parc Industriel Sidi Bou Othmane, 43303, Morocco

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