STERILE HYPODERMIC SYRINGE (WITH/WITHOUT NEEDLE)

Product Type

Hypodermic Syringe With/Without Needle	Needle Size	Basic UDI-DI : 611800222HYPOSRRGD
Luer 3 parts Syringe 1 ml, 3 ml, 5 ml, 10 ml, 20 ml, 50/60 ml	18G 21G 22G	
	23G 24G 25G	

Intended Use

- Sterile Hypodermic Syringe for single use intended for the aspiration of fluids or the injection immediately after filling, it is most commonly used for the administration of medications or for blood sampling.

Product Components

Name	Raw material
Barrel	PP - Polypropylene
Plunger	PP - Polypropylene
Gasket	Thermoplastic Elastomer, Latex free
Hypodermic Needle	Stainless Steel, SS304, PP - Polypropylene
Packaging	Medical grade paper and transparent film for EtO (individually blister packed)

Instructions for Use

- Inspect package for integrity and expiry and then remove the syringe from package using aseptic technique.
- Select proper size of syringe & needle for injection.
- Carefully select and clean the venipuncture or muscle site.
- Be careful to pull the needle cap straight off to avoid the needle point damage.
- Push and twist the needle clock wise on the syringe tip to seal it firmly and ensure a tight fitting.
- Draw the prescribed dosage into the barrel.
- Push the plunger carefully to take out any air bubbles.
- Inject the patient carefully with the needle.
- Push the Plunger thoroughly to the end of complete injection.
- Withdraw needle from patient.
- Do not recap the needle.
- Pull plunger to full extended position and broke it to prevent any one to use it again.
- Discard the needle after single use to prevent any one to use it again

The electronic version of "IFU/HYPOSRG/001 - Instructions for use" is available at www.immedicare.mq.

STERILE HYPODERMIC SYRINGE (WITH/WITHOUT NEEDLE)

Compatibility with other medical devices

Suitable for use with connectors "EN ISO 80369-7 Small-bore connectors for liquids and gases in healthcare applications - Part 7: Connectors for intravenous or hypodermic applications".

Indications

- Hypodermic syringes with needle are devices used to administer medical products and liquids.
- Syringes are designed to be used for intravenous, intramuscular, subcutaneous and taking blood samples.
- The product shelf life is 5 years.

Contraindications

- Hypodermic syringes with needle should not be used in more than one patient. The product can be used in a one-time sterile condition with a single patient.
- Cleaning or re-sterilization is not possible. It is not intended for use other than its stated intended use.
- Dispose of as medical waste after use.

Cautions

- Do not use the product if the package is damaged.
- Visually inspect that the content of package is complete and the package is undamaged.
- Do not use the product after the expiration date printed on the primary packaging.
- After injection or aspiration, dispose of the product as medical waste.
- The product is for single use only. It cannot be reused or re-sterilized.
- Do not touch the needle tip to prevent contamination/ needle sticks. Skin puncture with a contaminated needle may result in serious illness such as hepatitis, AIDS and known/unknown diseases.
- If the sterile packaging is not used immediately after opening, there is a risk of contamination. Product sterility cannot be maintained, do not use the product.
- Reuse of the product may cause infections, cross-contamination and sepsis.
- Sterile syringes are designed to be used immediately after filling, so do not store the medicine for a long time.
- If any serious incident occurs, it should be reported to the manufacturer and the competent authority

Storage Conditions

- Store in a cool and dry place.
- Keep it away from direct sunlight and moist environment.
- Do not place more than 5 successive boxes on the top of each other.
- The shelf life of product is 5 years.

STERILE HYPODERMIC SYRINGE (WITH/WITHOUT NEEDLE)

Description of Symbols Used

	Manufacturer		Fragile. Handle with care
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
	Batch code		Upper limit of temperature (30°C)
	Catalogue number		Humidity limitation (35% – 65%)
	Medical Device		Do not use if package is damaged and consult instruction for use
	Unique device identifier		Single sterile barrier system
	European Authorized Representative		Do not reuse
	Sterilized using ethylene oxide		Latex free
	Do not resterilize		Non-pyrogenic
	Caution		Consult instruction for use

Manufacturing Site

IM MEDICARE

Address: Lot 21, Parc Industriel Sidi Bou Othmane, 43303, Morocco

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