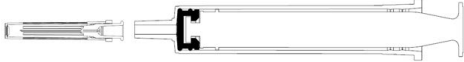
INSULIN SYRINGE

Product Type

Insulin Syringe	Needle Size	Basic UDI-DI : 611800222SYRINSBL
Luer 3 parts Syringe 0.5 ml, 1 ml	29G 30G 31G	

Intended Use

• A sterile device intended to administer insulin injection to a patient subcutaneously. It can be used by healthcare personnel or patients. It will allow for the aspiration of the pharmaceutical from its container for direct administration to the patient subcutaneously. This is a single-use device.

Product Components

Name	Raw material
Barrel	PP - Polypropylene
Plunger	PP - Polypropylene
Gasket	Thermoplastic Elastomer, Latex free
Hypodermic Needle	Stainless Steel, SS304, PP - Polypropylene
Packaging	Medical grade paper and transparent film for EtO (individually blister packed)

Instructions for Use

- Inspect the package for integrity and expiry. Remove the insulin syringe from the package.
 - To expose the needle, twist the protective cap to break the seal. Pull the cap straight off, being careful not to bend the needle or damage its point.
 - To measure the correct dose, align the top edge of the plunger with your dosage on the insulin syringe scale.
- After injection, destroy the syringe needle and place it in a container for safe disposal to prevent misuse.
- Discard the insulin syringe after one use. It is extremely difficult to clean properly after exposure to biological materials, which may cause adverse patient reactions if reused. Cleaning may alter the device's structural integrity.
- The electronic version of "IFU/SYRINS/001 - Instructions for use" is available at www.immedicare.ma.

INSULIN SYRINGE

Steps for Using the Insulin Syringe

Step 1: Open package by peeling back paper and remove syringe carefully.

Step 2: Remove protective needle sheath/cap carefully to avoid needle point damage.

Step 3: Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.

Step 4: Once the insulin syringe used, immediately dispose of syringe in an approved sharps disposal Container.

Compatibility with other medical devices

Suitable for use with connectors "EN ISO 80369-7 Small-bore connectors for liquids and gases in healthcare applications - Part 7: Connectors for intravenous or hypodermic applications".

Indications

- Insulin syringes with needle are devices used to administer medical products and liquids.

Contraindications

- Device must not be used other than as indicated in the intended use.
- Device should not be used for the administration of blood and blood components.
- Device should not be used for large volume fluids and fluid derivatives.
- Device should not be used for the administration of high viscous fluids.
- Device should not be used on patients with known hypersensitivity to device material.

Cautions

- Device is for single use only.
- Device must not be reused. Reuse of the device may cause serious infection, cross-contamination, and transmission of bacteria, viruses, and blood-related diseases.
- Ensure before use that the device/packaging is not tampered with or damaged.
- Device is for manual use only.

Storage Conditions

- Store in a cool and dry place.
- Keep it away from direct sunlight and moist environment.
- Do not place more than 5 successive boxes on the top of each other.
- The shelf life of product is 5 years.

INSULIN SYRINGE

Description of Symbols Used

	Manufacturer		Fragile. Handle with care
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
LOT	Batch code		Upper limit of temperature (30°C)
REF	Catalogue number		Humidity limitation (35% – 65%)
MD	Medical Device		Do not use if package is damaged and consult instruction for use
UDI	Unique device identifier		Single sterile barrier system
EC REP	European Authorized Representative		Do not reuse
STERILE EO	Sterilized using ethylene oxide		Latex free
	Do not resterilize		Non-pyrogenic
	Caution		Consult instruction for use

Manufacturing Site

IM MEDICARE

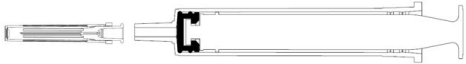
Address: Lot 21, Parc Industriel Sidi Bou Othmane, 43303, Morocco

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